## Active Gogging Shoe QUICK STEP- installation



1. Open the shoe wide





2. Place the shoe on the hoof. Make sure that the toe is all the way in front of the shoe, and that the shoe is centered in the back.



3. Put the Velcro together. Check that both Velcro's are aligned on top of each other.



4. Alternatively, cross the Velcro downwards to get a better fit around the pastern.



5. Pull the strap firmly with a hoof pick and secure. This will pull the shoe more tightly around the hoof.



6. Close front cover and secure the smart lock on the side in the hole which fits your horse.



7. Ready to run!